



Professional Resources

Confirm Your Clinical Diagnosis With... **CARIES DETECTOR**

Photographs by Jeffrey C. Hoos, D.M.D., F.A.G.D.

prob•lem ('prāb-ləm) *n*
Detecting Outer Caries



so•lu•tion (sə-'lu-shən) *n*

There are two layers of carious dentin! Do you know how to detect them properly?

- THE ORIGINAL CARIES DETECTOR
- NEW QUANTITY AND LESS COST
- CONSERVES TOOTH STRUCTURE
- DECREASES NEED FOR LOCAL ANESTHESIA

In addition to your expert clinical judgment, Caries Detector confirms your clinical diagnosis and ensures all the caries is removed. **CARIES DETECTOR** by J. Morita USA, Inc. is the original caries detecting material with 18 years of clinical use. It clinically differentiates between the tissue that should be removed from that which should be preserved. Pictured below are four examples of the many clinical uses of **CARIES DETECTOR**.



Decalcification or Caries?



Application of Caries Detector

Home
Teeth For A Lifetime
Balancing
Articles
Seminars
Brush & Floss Dental Center
Contact Us

© 1999 - 2005
Dental Explorations LLC

J.MORITA
USA
www.jmoritaua.com

BASIC
Dental Implants LLC

Visit
Dr. Hoos'
Practice
Web Site
Brush & Floss
&
Dental Center



Cervical caries detected only on #28



Traditional crown preparation appears to be caries-free



After application: Visible caries detected, preventing possible sensitivity after crown cementation



After complete caries removal



Symptom: biting pain



Close-up of only restoration in patient's mouth



Fractures visible after application of Caries Detector



Patient symptoms:
Hot and Cold sensitivity



Restorations removed and Caries Detector applied



initial caries detected at first application



2nd application: more decay found on walls of preparation



Final preparation: caries free



Caries Detector 2 x 6ml bottles.

[This article is available in PDF Format](#)

[More information on Caries Detector](#)
available from **[J. Morita USA, Inc.](#)**



[Return to Main Page](#)

Dental Explorations L.L.C.

THE INFORMATION SOURCE FOR ALL DENTAL PROFESSIONALS



Betsy Smirnoff Hoos, Director

**8 Hollow Oak Road
Woodbridge, CT 06525**

**Phone: (203) 389-6263
Fax: (203) 397-3632**

deinfo@dentalexplorations.com



Designed and hosted by

SK web construction